


# Alcovisor<sup>®</sup>

## Beta

Fuel Cell Breath Alcohol Tester

# Instruction Manual

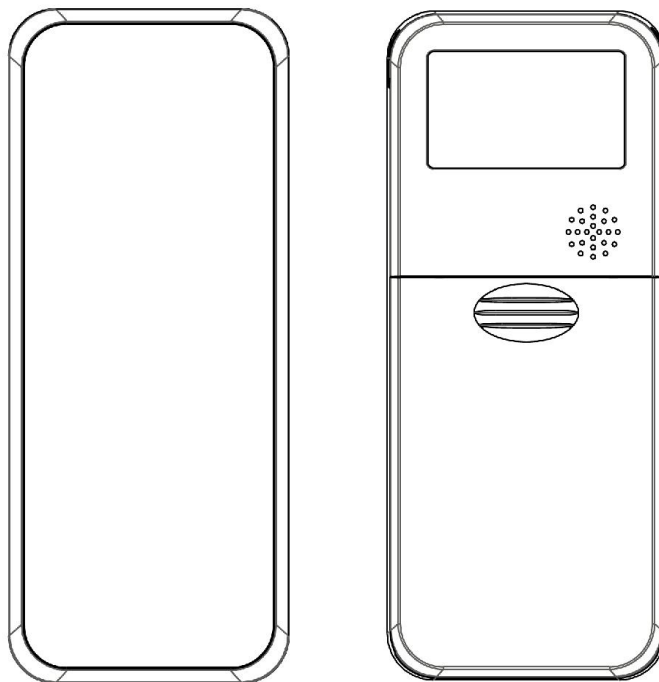

Website: <http://www.alcovisor.com>

Thank you for purchasing Alcovisor® products. Please read this instruction carefully before using the products and follow the instructions when you are using the product.

## Points to Remember

1. There must be an interval of at least 20 minutes after alcohol has been taken into the mouth. Residues of alcohol in the mouth may give false measurements. Such residues may be left by aromatic drinks, mouthwash, medicines and drops, as well as by belching and vomiting. Rinsing out mouth with water or non-alcoholic drinks do not substitute for an interval.
2. Some foods and even “non-alcoholic” drinks may contain traces of alcohol, which will affect the test result through a “mouth alcohol” effect. To prevent this, wherever possible, ensure that a delay of about 20 minutes has elapsed since taking anything by mouth – even medicines which may contain alcohol. Even a glass of water prior to the test will cool the mouth and dilute the saliva, temporarily reducing the amount of alcohol in the breath, and, consequently, the instrument reading. A delay of at least two minutes should take place between the time of last smoked and the test.

## Operating Instruction

1. Insert 2 pcs of AA alkaline batteries.
2. Press and hold the On/Off button for 2 seconds to switch on the device.
3. The measurement unit of this device is displayed.
4. Press the On/Off button, the word “BLO” is displayed. Insert a mouthpiece, take a deep breath and blow to the device CONTINUOUSLY. A beeping tone will be heard and the “0” will be flashing on the LCD. Test result will display on the LCD after a click sound. If there is no blowing within 2 minutes, the device will switch off automatically.
5. To make a passive test, press the On/Off button during the blow.
6. Press the On/Off button once to perform another test.  
**Reminder: To perform a test immediately after a POSITIVE test, change the mouthpiece and make a passive test with clear air before the test to prevent residues.**
7. Press and hold the On/Off button for 5 seconds to switch off the device. The device will also be automatically switched off after 5 minute.